


Alternating Pressure Relief System User Manual



Arise B BUBBLE PAD Specifications	
Dimension (LxWxH)	200 x 94 x 7 cm
Material	Medical grade PVC - 0.29 mm thickness
Weight	1.95 KG
Weight Capacity	100 KG / 220 lb
Features	<ul style="list-style-type: none"> 130 individual air bubbles for an even support and weight distribution For prevention of low risk pressure ulcers on short term usage Extension flaps to secure bubble pad to existing mattress

Arise B PUMP Specifications	
Power Supply	220V/50Hz; 7W
Pressure Range	50 - 105 mmHg
Air Output (min)	4 - 6 liters
Cycle Time	10 min
Size (LxW xH)	25.8 x 10.2 x 11.7 cm
Weight	1.3 KG
Features	<ul style="list-style-type: none"> Low vibration silent technology User weight indicator to facilitate optimize pressure setting Quick-change external air filter Low power consumption

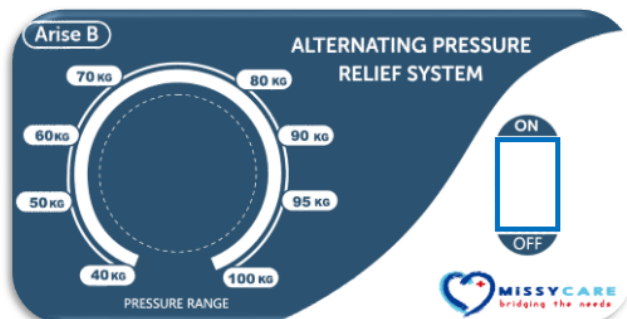

CAUTION!

- Do not use the product for a purpose not indicated in this manual.
- Only qualified personnel or trained for the treatment and prevention of bed sore can operate with the device.
- The manufacturer reserves the right to change the information contained in this document without previous notice.

INTRODUCTION

Thank you for purchasing MissyCare Arise B Bubble pad and pump with pressure adjustment system. This user manual provides guidance on how to use the product correctly. Please read through the manual carefully before using the product. Should you have any further queries, please contact your retailer for advice and assistance.


NOTE Pictures are only for illustration purposes and may not refer to a specific model

GENERAL DESCRIPTION

Bubble pad and pump

INTENDED USE

The pump and bubble pad system are designed for the prevention of bed sore which may occur during prolonged bed confinement.

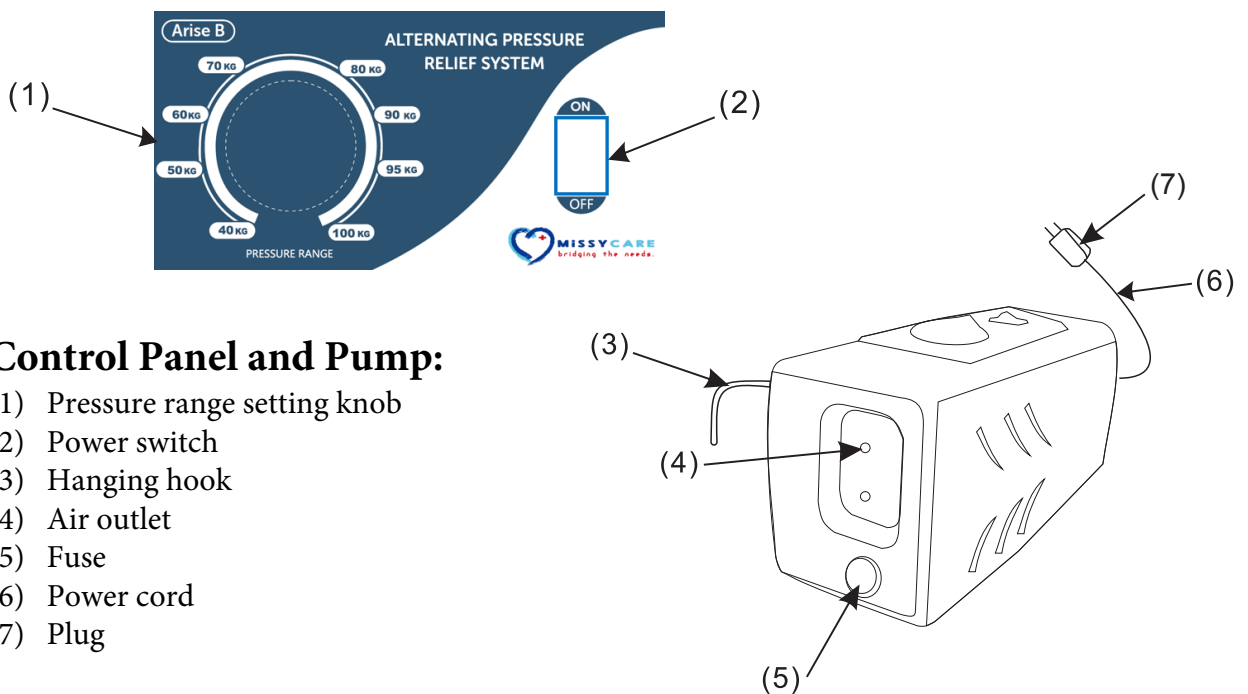


Control Panel and Pump:

- (1) Pressure range setting knob
- (2) Power switch
- (3) Hanging hook
- (4) Air outlet
- (5) Fuse
- (6) Power cord
- (7) Plug

Bubble Pad:

- (1) Head extension flap
- (2) Air cells
- (3) Lot number
- (4) Air outlet
- (5) Air tubing
- (6) Foot extension flap

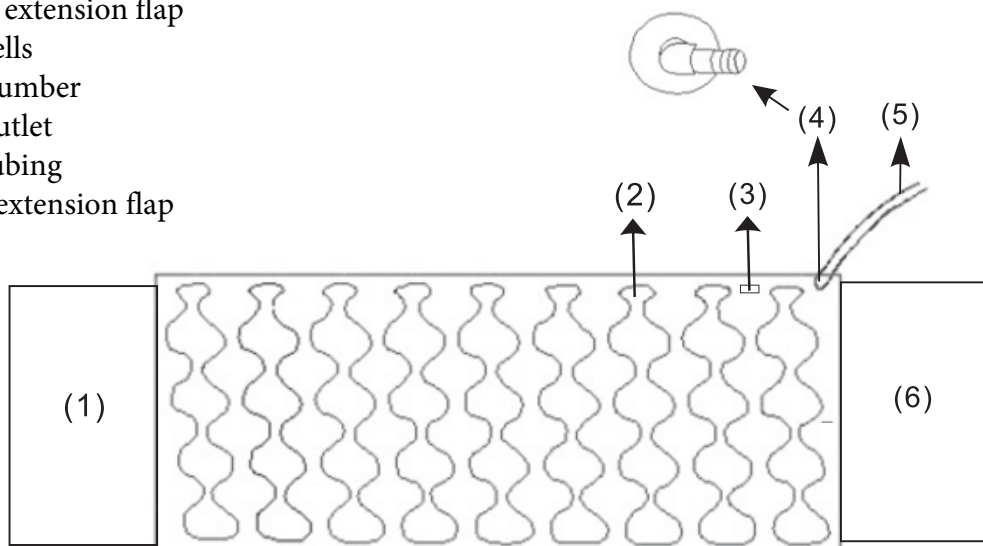

GENERAL WARNINGS


1. DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual. If you are unable to understand the warnings, cautions or instructions, contact a healthcare professional, dealer or technical personnel before attempting to use this equipment.
2. Consult physician or therapist to determinate the correct adjustment and the correct use of the device.
3. Keep the motor in well ventilated environment away from heat sources.
4. DO NOT stand on the bubble pad as it may damage air cells prematurely.
5. DO NOT allow children to play or operate the pump.

INSTALLATION


1. Place the bubble pad on a common mattress with the air tubing connectors oriented to the footboard.

NOTE Place the bubble pad on a 5cm common mattress (recommended)

2. Tuck the bubble pad to the common mattress underneath using the extension flaps. Hang the pump onto the bed rail (footboard side).

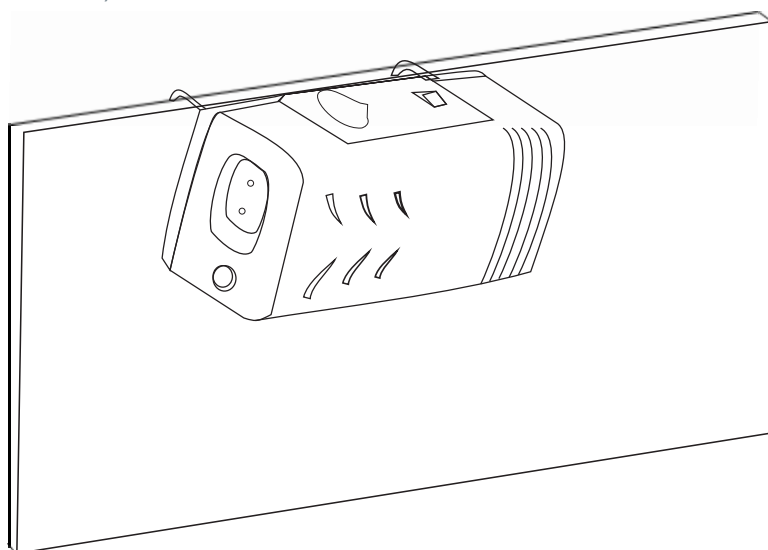

3. Connect air tubing from bubble pad to the air outlet on pump unit. Ensure the air tubing are free from entanglement.
4. Plug the power cord into the electrical outlet. Turn the switch ON.
5. Adjust the pressure knob accordingly to the user weight.

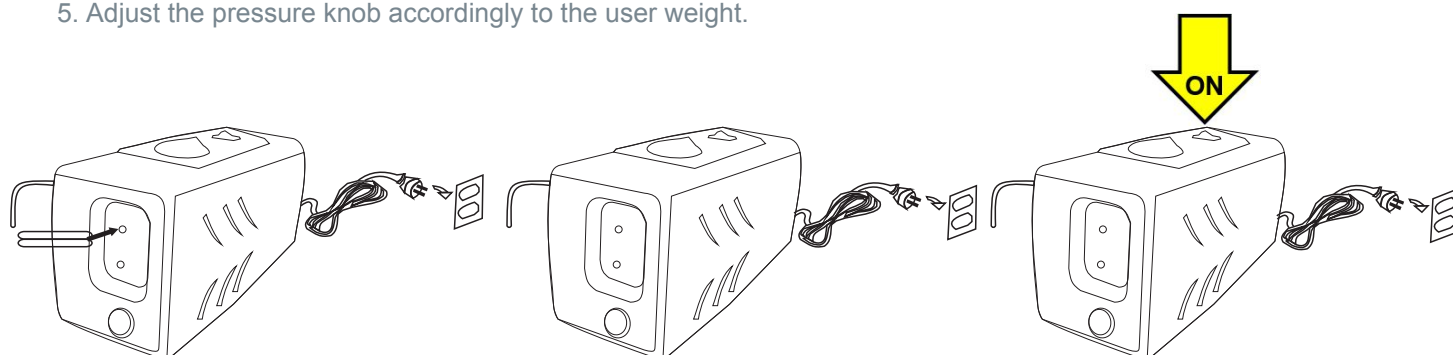

NOTE In case the pressure is constantly low, check for any holes (on the interchangeable elements, on the connection tubes or bubble pad). If it is necessary, you may patch the damaged bubble pad using provided patch kit, otherwise contact your dealer.

CLEANING AND DISINFECTION



WARNING!

Unplug the pump before starting the cleaning procedure.

1. Clean the bubble pad with mild soap.
2. Do not immerse or soak pump unit.
3. After cleaning, dry the bubble pad without direct sunlight exposure.
4. Ensure the connection tubes are not covered by dust. In this occurrence, use a soft swab to clean the tubes

NOTE DO NOT use acid, alkaline or solvent such as acetone

5. Clean and dry the external cotton filter periodically to maintain the efficiency of pump.
Change the cotton filter whenever necessary.

TROUBLESHOOTING

If your questions can't be resolved using the information below, please contact your local retailer for further advice.

No lights on unit	<ul style="list-style-type: none"> - Check control unit is connect to the mains power supply and that the unit is switched on. - Check control unit fuse or main power plug fuse.
Constant low pressure	<ul style="list-style-type: none"> - Check main power to the control unit and that the plug is switched ON. - Check air supply connectors are connected to the pump unit correctly - Check the connector tubes for any entanglement.
User's body is sagging in the middle/ bottoming out	<ul style="list-style-type: none"> - Increase the pressure range on the pump. Wait a couple of minutes so pressure can stabilize before making further adjustment.
Control lock up or "freeze"	<ul style="list-style-type: none"> - Turn the control unit off and unplug the pump. Wait for a few minutes and plug the control unit back in. Turn on the control unit.

WARRANTY

MissyCare products are guaranteed from material or manufacturing faults for 12 months from the purchasing date, except possible exclusion or restriction as follows:-

1. The warranty shall not be applied in the possible damages caused by improper use, abuse or alteration, and the warranty will not be valid if the instructions for use are not strictly adhered to the correct intended use is specified in this manual.
2. **Plexus Care Pte. Ltd.** declines all responsibilities for any consequences such as damage, personal injuries or whatever caused by resulting from an incorrect/improper use or in relation to wrong installation of this product and from unauthorized alteration to the frame of the product.
3. **Plexus Care Pte. Ltd.** warranty does not cover damages resulting from: natural disaster, not authorized maintenance or repairs, faults caused by problems on electricity supply (when necessary), use of spare parts not covered by Plexus Care Pte. Ltd., improper use, unauthorized alteration to the frame of the product, or in case of insufficient maintenance as indicated in the manual.
4. The warranty doesn't cover components subject to wear and tear during the correct use of the device.
A repair or a replacement doesn't extend the warranty.

NOTE Proof of purchase must be presented in soft or hardcopy for any warranty claim.